

## EZ ROM Elite Post-Op Brace

The EZ ROM Elite Post-Op Adjustable Knee Brace allows the patient to adjust the length of the EZ ROM Elite for a complete, custom fit. The brace is ideal for knee ligament damage, knee surgery, and instability of the knee. The EZ ROM Elite is designed for post-operative fixation; allow the physician to lock the patient into extension and control range-of-motion as prescribed.



### SPECIFICATIONS

- Light weight and breathable material
- Latex free design
- Prefabricated, Off-the-Shelf design
- HCPCS Code L1833

### FEATURES & BENEFITS

- Telescoping and malleable hinge bars extendable from 16.5" to 27"
- Range of Motion adjustable – Flexion (0° to 120°); Extension (0° to 90°)
- Quick lock key offers ability to control & lock out hinge motion to 0°, 15°, and 30° directly.
- Easy application w/hook & buckle closure

### INDICATIONS FOR USE


- ACL, PCL, MCL, LCL repairs, injuries
- Tibial plateau fractures
- Patella tendon repairs
- Osteochondral repairs
- Meniscal repairs
- Stable femoral fractures



Safety Lock



Hinge with quick lock feature

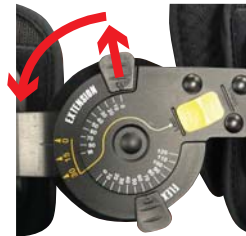


#### Extension

To control Extension, pull the extension tab out and move to desired setting between 0° and 90° and release tab to lock into position.



#### Flexion

To control Flexion, pull the Flexion tab out and move to desired setting between 0° and 120° and release tab to lock into position.

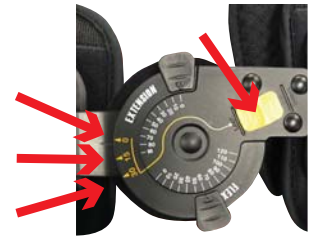

#### Quick Lock

The hinge can be locked out at 0°, 15°, 30° by sliding the yellow tab into the locked position.



# ThermoTek™

1200 Lakeside Parkway, Flower Mound, TX 75028

