

The Kahuna Brace provides support for shoulder surgeries in a universal, multiple configuration brace. Kahuna is suitable for all types of shoulder surgery; as well as conservative care of shoulder injuries. The Kahuna Brace is the physician's new "go to" shoulder brace.



SPECIFICATIONS


- Light weight and breathable material for patient comfort
- Abduction angles from 30° to 90°
- Latex free materials
- Prefabricated, Off-the-Shelf design
- HCPCS Code L3960 (when fitted with airplane control bar)

FEATURES & BENEFITS

- Universal sizing; Left or Right
- Low-profile design w/adjustable sling
- Two step application
- True airplane or gunslinger alignment
- Multiple abduction angles
- Extreme external rotation

INDICATIONS FOR USE


- Rotator Cuff Repairs
- SLAP Repairs
- Bankart Repairs
- Anterior subluxation
- Joint reconstruction
- Shoulder instability

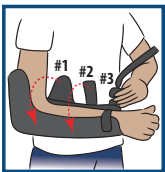

1. Open hook and loop strap #1 and #2, also open buckle #3 on the arm sling. Place arm in the brace and secure both hook and loop strap #1 & #2 comfortably around arm. To fit your arm into the sling you can fold the sling along the pre folded point under the arm sling buckle #3.



3. Position the back support pad #6 in the middle of the patients back. Then adjust the torso strap by adjusting the belt loop #7 which is at the base of the buckle #8.

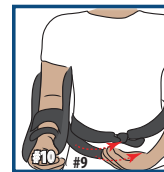


5. Adjust the abduction toggle bar #9 and the moldable arm bar #10 to desired position. Increase abduction by pulling the abduction toggle bar #9 towards the body.



2. Place shoulder belt #4 around the opposite shoulder and adjust the sling #5 to fit comfortably around the shoulder.



4. Position the abduction toggle bar #9 on the injured side of the body and position the moldable arm bar #10 pointing forward.

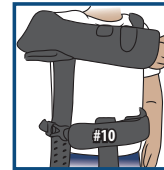

6. To adjust the Kahuna brace into the Airplane position, attach the Airplane control bar by removing the foam padding at the back of the moldable arm band #10. There are 2 holes to insert the Airplane control bar. Insert 2 screws (provided with Allen wrench)



ThermoTek™
1200 Lakeside Parkway, Flower Mound, TX 75028

