

The Mini Kahuna Brace provides support for shoulder surgeries in a universal configuration brace. The Mini Kahuna Brace is universal and can be applied to either the right or left side. The Mini Kahuna Brace features a breathable material which allows for comfort and optimal patient experience.

The Mini Kahuna brace comes complete with abduction pillow and ball.



SPECIFICATIONS

- Light weight and breathable material for patient comfort
- Latex free materials
- Prefabricated, Off-the-Shelf design
- Designed for wicking moisture away from the body
- HCPCS Code L3670

FEATURES & BENEFITS

- Universal sizing; Left or Right
- Low-profile design w/adjustable sling
- Simple patient application
- Gunslinger alignment
- Abduction pillow w/ball
- Quick release shoulder strap

INDICATIONS FOR USE


- Rotator Cuff Repairs
- SLAP Repairs
- Bankart Repairs
- Anterior subluxation
- Joint reconstruction
- Shoulder instability



1. Open hook and loop strap #1 and #2, also open buckle #3 on the arm sling. Place arm in the brace and secure both hook and loop strap #1 & #2 comfortably around arm. To fit your arm into the sling you can fold the sling along the pre folded point under the arm sling buckle #3.



3. Position the back support pad #6 in the middle of the patients back. Then adjust the torso strap by adjusting the belt loop #7 which is at the base of the buckle #8.

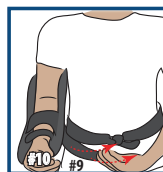

5. Make final adjustments for comfort. Adjust periodically if needed.



2. Place shoulder belt #4 around the opposite shoulder and adjust the sling #5 to fit comfortably around the shoulder.



4. Position the abduction pillow #6 on the injured side of the body.

Warning: The Mini Kahuna brace is supportive only and does not guarantee or intend to prevent injury. Consumer or patient agrees to the following.

- ThermoTek accepts no responsibility for any injury which might occur while using ThermoTek products.
- Skin irritation may occur while using this device. If you experience an unusual skin reaction while using this product, consult with your healthcare provider immediately.
- Consult your health care provider for this products indication for use.
- For single patient use only.
- Federal Law restricts this device to sale or by or on the order of a licensed health care practitioner.
- If you do not agree to the above conditions, contact your health care provider before using this product.

Washing and Care:

The Mini Kahuna brace may be cleaned by hand using mild soap and warm water. Hang to dry. Do not place in washing machine or dryer.



ThermoTek™
1200 Lakeside Parkway, Flower Mound, TX 75028

